


ANX AESTHETINEXT

DERMATOLOGY & BEYOND

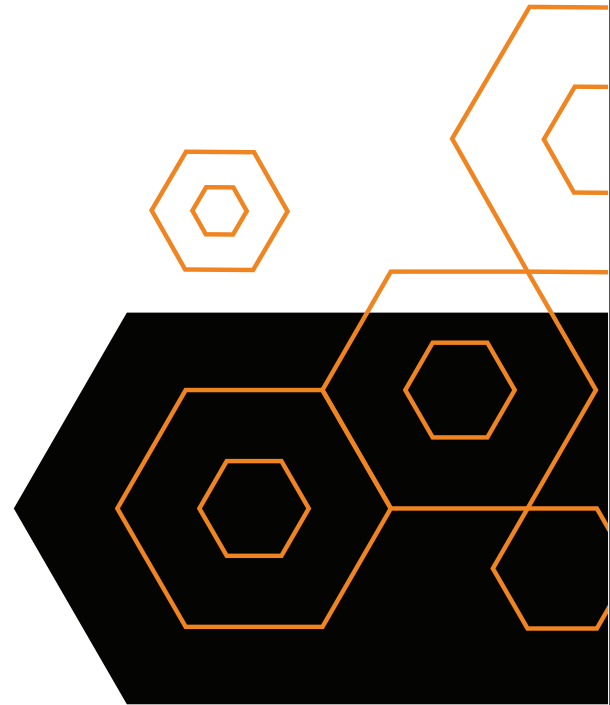

SKIN

BODY

HAIR

SEXUAL WELLNESS

CONSUMABLES

SCAN ME



Our Story

At Aesthetinext, we decode the needs of practitioners and the aspirations of their patients to engineer smarter aesthetic solutions. Every innovation we create stems from intelligent insight, advanced research, and real-world feedback — transforming clinical precision into technological excellence. This seamless blend of science and design defines who we are, and the future we're shaping in aesthetic medicine.

Vision

To lead the evolution of aesthetic technology by uniting medical science, engineering, and design — creating a future where innovation amplifies every practitioner's touch.

Mission

To design intelligent, data-driven aesthetic technologies that empower practitioners with precision, reliability, and performance — making advanced skincare accessible, effective, and safe.



EM Sculpting Pro

5000W STRONG POWER

4 HANDLES WORK SIMULTANEOUSLY

Emsculpting pro has a wide range of application scenarios, which can be used in beauty clinics, gyms and even offices or homes to shape your perfect body anytime, anywhere.

SPECIFICATIONS

Frequency:	1-150hz
Power:	5000w
Intensity:	7Tesla
Screen:	12.4inch tough screen
Pulse width:	310 μ s \pm 10%
Cooling:	Air Cooling
Voltage:	AC220V \pm 10% 10A 50HZ , 110v \pm 10% 10A 60HZ



The EMS BODYSCULPTING PRO is a fast, effective and innovative solution:

An accelerated physical transformation

The results are visible from the first session and complete about 6 to 8 weeks after the last session.

Optimised performance

A 30-minute session induces 24,000 supramaximal muscular contractions, the equivalent of 3 to 5 hours of gym work for squats or abdominal exercises.

A toned and firm body

The studies carried out show, at the end of a protocol of 4 sessions, an average reduction of - 19% of fatty tissue, as well as an average increase of + 16% of muscle mass in the treated area.

Several possible treatment areas

Buttocks : stimulation of all the muscles (small, medium and large) for a rounded and shapely buttock (+15% buttock lift - booty lift effect)

Abdominals : strengthening of the abdominal belt (rectus abdominis to obtain a well-defined stomach, transverse muscles for a flat stomach and obliques for a V-shaped and sheathed stomach)

Arms and thighs: stronger and firmer muscles

Non-invasive treatment

The EMS BODYSCULPTING PRO acts simultaneously on fat masses and muscle architecture. It is therefore particularly suitable after pregnancy to reduce the spread of the abdominal muscles (correction of the diastasis of the rectus abdominis post-partum)

SMAS Focus V HIFU

1200W STRONG POWER

NON-INVASIVE AESTHETIC TREATMENT

The working principle is to focus high-intensity focused ultrasound on a point to generate high energy, which acts on different levels of the skin, stimulates the proliferation and reorganization of collagen and reduces fat, achieve the effect of firming contours and smoothing lines and shaping. There are two main points of HIFU treatment: depth and temperature.

SPECIFICATIONS

Frequency:	4 ~ 7MHz
Power:	1200w
Net Weight:	25 kg
Screen:	12.4inch tough screen
Dimension:	400*350*1060mm
Voltage:	100-240v 50/60Hz



This advanced technology is suitable for:

People who want a non-invasive, scalpel-free face lift

Reducing skin laxity and redefining facial and body contours

Removing localized fat in areas such as the abdomen, thighs, arms, double chin

A safe, fast and effective treatment with no recovery time



Vmax single dot handle



1.5mm, 3.0mm, 4.5mm



2D hifu handle



1.5mm, 3.0mm, 4.5mm



3D hifu handle



1.5mm, 3.0mm, 4.5mm

Eyebrow, Wrinkles around eyes, Wrinkles around month

Check, Forehead, Double chin, Neck wrinkles, V-line forming

Check, Forehead, Double chin, Neck wrinkles, V-line forming

Lumineau 10-in-1 Hydrafacial

1200W STRONG POWER

NON-INVASIVE AESTHETIC TREATMENT


10 IN 1 Hydra Facial Machine | HydraFacial Multifunctional Machines a preferred instrument for facial care, offering eight functions in one multi-functional skin care device. With its new upgrade, each handle provides unique effects, leaving the skin feeling refreshed, clean, and delicately treated after each session.

SPECIFICATIONS

Voltage	100V - 240V 50Hz - 60Hz
Power	<350W
RF Frequency	Biopolar, quadripolar
Cold and hot Scope	0 - 45 C
Ultrasound	1.1MHz
Probe	4 soft heads and 8 hard heads
Hydrodermabrasion handle	90Kpa pressure. Flow capacity: 30L
Positive Pressure	3-4kg
Display	8inch
Solution bottle	A. Deep layer cleaning B. Whitening and Moisturizing C. Fruit acid rejuvenation D. Pipe cleaning



Includes A Comprehensive Range Of Effective Facial Transforming Treatment

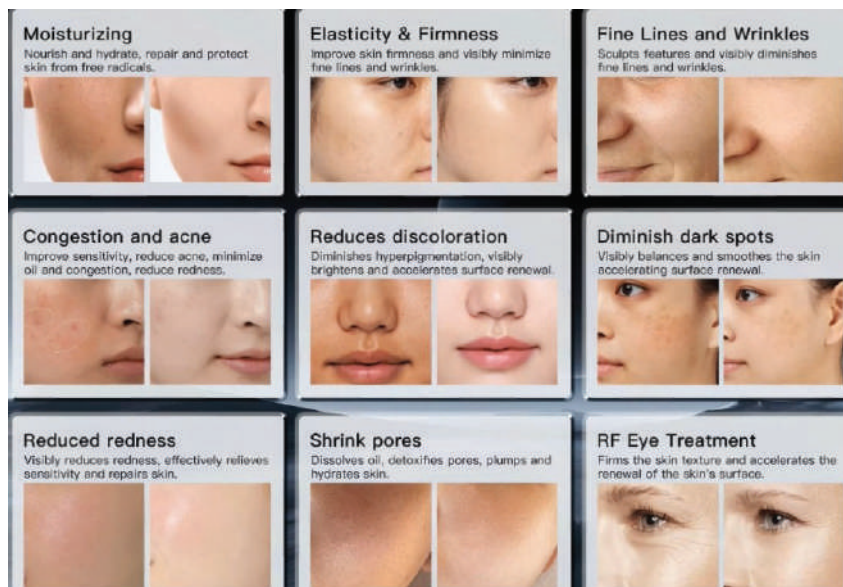

Fractional CO2 Laser

70 W HIGH POWER

USA IMPORTED NANO SENSOR MOTOR

The technology utilises laser energy microbeams to penetrate and break down skin tissue through the epidermis. The laser beam will target into the dermis and create tiny wounds throughout that particular area without damaging the tissues in the surrounding parts. Your body will naturally heal and during the process, creates new, healthy tissue to replace the affected areas that have undergone treatment. Scars should be of a lighter colour and your skin texture will be more even, leaving no visible wounds and allows you to apply make-up immediately following the procedure.

SPECIFICATIONS

The type of laser	10600nm Carbon dioxide laser
Focus spot diameter	≤0.1mm
Maximum Output Power	70w
Pulse width	0.1~10ms(0.1ms step)
Repeat time	1~20 times (1 step)
Scan Shape	Square; Rectangle; Circle; Triangle; Hexagon; Ellipse; Line
Maximum scan area	20×20mm
Scan Patterns	Standard; Random; Scatter
Screen	12.1" Inch LCD Touch Screen
Voltage	AC 220V 50Hz ;AC 110V 60Hz
Dimensions	550mm*480mm*1200mm
Net Weight	66kg



Wrinkle removal



Acne scar removal



Stretch mark removal



Large pore removal

Picosecond Nd Yag Laser System

1064nm 532nm (585nm, 650nm optional)

MOST ADVANCE , COST EFFECTIVE ACTIVE NDYAG

The Pico Nd Yag laser delivers specific-wave lengths of light extremely quickly (in pico-seconds) and at a very high energy. These pulses are selectively absorbed by the tattoo pigment, shattering it into tiny fragments small enough for the body to recognise the particles as a foreign object, and over the next several weeks some of particles (in shallow cuticle) are discharged from the body and the other part (in deep tissue) are absorbed and digested by the body's macrophages and removed out of the body by the immune system. Since the specific wavelengths used can only be absorbed by pigment particles, usually no lasting injury happens to normal skin and normal tissues.



SPECIFICATIONS

Laser Type	Solid State Picosecond Nd:YAG
Wave Length	1064nm 532nm (585nm,650nm optional)
Energy	Max1500mj(1064nm);Max800mj(532nm)
Frequency	1~10Hz
Zoom Spot Size	2-10mm Adjustable
Pulse Width	2000ps
Beam Profile	Top Hat Beam/ Fractional Beam
Light Guiding System	South Korea 7 joints Arm
Aiming Beam	Diode655nm(Red),AdjustableBrightness
Cooling	Closed Circuit Water to Air
Voltage	AC220v±10% 50Hz,110v±10% 60Hz
Net Weight	87 kg

ULTRA SHORT PULSES

MULTI WAVE LENGTHS

HIGHER PEAK POWER

Types of Treatment

Melasma

Age Spot

Birthmark

Nevus Ota

Freckles

Seborrheic Keratosis

Skin Rejuvenation

Post-acne erythema

Inflammatory acne

Rosacea

Diolasheer ICE - 2000 W

ULTRA FAST COOLING, TRIPPLE WAVE LENGTH

LATEST DOUBLE HANDLES DIODE LASER HAIR REMOVAL

The principle of diode laser hair removal treatment is based on selective photother - mal decomposition. The melanosomes in the hair follicle can selectively absorb the energy of the laser. The laser energy emitted by the machine is easily absorbed by the colored hair follicle without damaging the epidermal tissue. The energy emitted by the laser is absorbed by the hair and the pigment in the hair follicle. It is converted into heat, thus increasing the temperature of the hair follicle. When the temperature rises to a certain level, the hair follicle will be irreversibly damaged, and the hair will lose its original environment and be completely removed.

SPECIFICATIONS

Power	3000W
Power of handle	600-2000W optional
Wavelength	808nm (755nm+808nm+1064nm optional)
Machine screen	12.1 inch
Handle screen	1.54 inch
Energy density	1-120J/cm2 (Deviation $\leq\pm 2\%$)
Pulse width range	1-200ms
Spot size	12*12mm ; 12*20mm; 12*24mm; 12*28mm
Frequency	1-10HZ (600-1200w) 1-20HZ (1600-2000w)
Cooling System	TEC cooling system
Net Weight	54kg
Dimension	512*541* 1303mm
Package Size	630*690*1210mm
Fuse specification	$\varnothing 5 \times 25$ 10A
Voltage	AC220V $\pm 10\%$ 10A 50HZ , 110v $\pm 10\%$ 10A 60HZ



Comfortable

Safe

Effective

Fast

TRIPLE WAVELENGTHS, BEST RESULT FOR ALL SKIN TYPES AND ALL HAIR TYPES

755nm


755nm offers more powerful energy absorption by the melanin chromophore, making it ideal for light colored and thin hair. It targets the Bulge of the hair follicle and is especially effective for superficially embedded.

808nm

808nm is classic wavelength in laser hair removal, the 808nm wavelength, offers deep penetration of the hair follicle with high average power, suitable for most skin types and hair types.

1064nm

1064nm wavelength is characterized by lower melanin absorption, making it a focused solution for darker skin types. At the same time, the 1064nm offers the deepest penetration of the hair follicle.

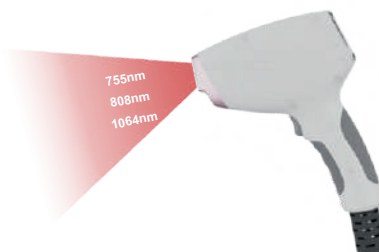

EMS Hydra Star

ULTRA FAST COOLING, TRIPPLE WAVE LENGTH HANDLE

LATEST DOUBLE HANDLES DIODE LASER HAIR REMOVAL

Nanocrystals are made of monocrystalline silicon. In addition to being tough, monocrystalline silicon can also be thin enough and short enough to only break through the barrier layer of epidermal absorption, and it's far from touching blood vessels and nerves, so you can't feel pain, nor will it bleed.

Nanocrystals promote the absorption of skin care products by opening the skin absorption channel, so skin care products can be more absorbed by the skin when nano crystals are introduced.



SPECIFICATIONS

Voltage	100-120 V / 60 Hz 220-240V / 50 Hz
Display	10.4 inches
Power	500 W
Heat & Cool	Heat : 36-46 Cool: 0-6
Aqua Peel Probes	Soft : 3pcs Hard 8pcs
Aqua Peel	Vaccum intensity: 4-5 kg Pressure : 96Kpa Water flow: <50L
Solution Bottle	A: Cleaning B: Whitening and Moisturizing C : Deep Layer Cleaning D : Pipe Cleaning
Cleansing Shovel Frequency	26-30MHz
EMS NANO Mesotherapy Size	4 X 4mm X 250Um
Micro Lift	Frequency : 200Hz Current : 50-800Ua
Sin Analyser	28 Million HD Mega Pixels

ANTI-ALLERGY

SOOTHING

REPAIR

SKIN TIGHTENING

INTELLIGENT OPERATING SYSTEM


- Versatile Functions
- Versatile Languages
- User - Friendly UI
- 10.4" display
- 10 in 1 Platform




Sagging skin



Dry and water shortage

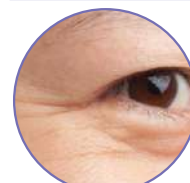

Fine water wrinkles

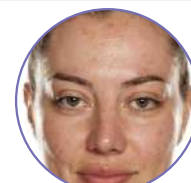

Gloomy

Vertical MNRF

FRACTIONAL RF MICRO NEEDLE MACHINE

LATEST DOUBLE HANDLES DIODE LASER HAIR REMOVAL

Fractional RF Microneedling MNRF is a beauty device combining perfect effects of Radio Wave and . Micro-needle Fractional Mesotherapy. Combination of those two methods in one treatment enhances the scars reduction. wrinkles and stretch marks removal. and guarantees quick texture improvement. The device is equipped with two heads which enables using two different methods of radio waves working.

Superficial Fractional RF has two unique channels which combine to provide epidermal and dermal coagulation. The first channel delivers a deeper zone of controlled thermal effect to activate collagen and skin enhance skin tightening. The second channel provides micro-ablation of the upper layers of the epidermis and mild coagulation for controlled fractional ablation and skin rejuvenation with minimally invasive RF delivery. Together, the two channels provide a 3-dimensional treatment zone for superior skin rejuvenation, overall tightening and consistent results you can count on.




SPECIFICATIONS


Frequency	5 MHz
Total RF Treatment Level	1-10
Power	80 W
Types of Needles	81 tips, 49 tips, 25 tips
Depth of Needle	0.3 - 3 mm (Adjustable)
Switch	Foot Switch
Dimensions (CM)	40 X 38 X 80 cm
Weight	20 kg

- Face Lift
- Wrinkle
- Skin Tightening
- Skin Rejuvenating
- Scar Remover
- Acne Treatment

Treatment Scope

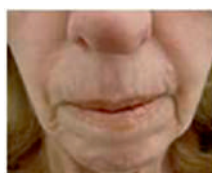

Before

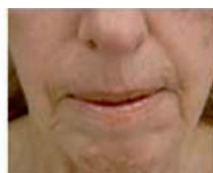


After



Before




After

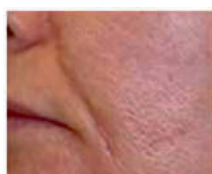


Before



After

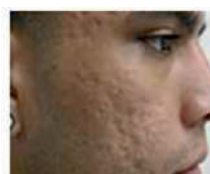

Before

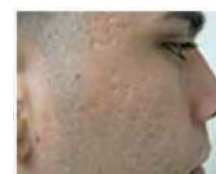


After

Vertical MNRF

FRACTIONAL RF MICRO NEEDLE MACHINE

User friendly operation system

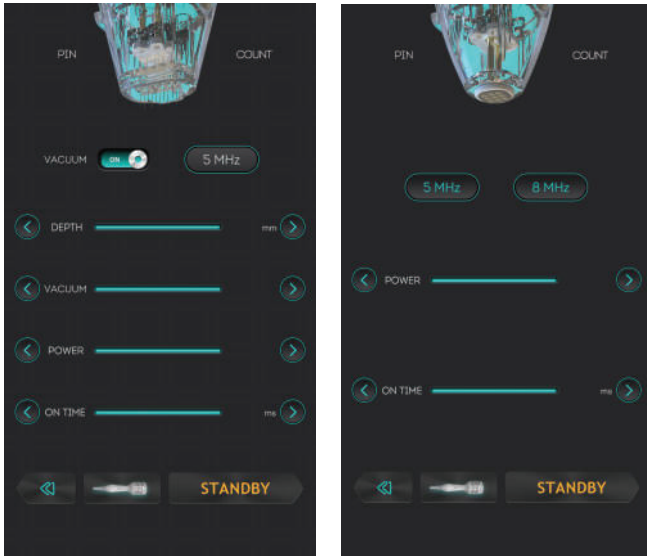

SPECIFICATIONS

Power	400W
RF Frequency	5MHz/8MHz
Screen	Screen: 12.4inch
Plus Width	Microneedling handle 250-600ms Thermagical handle 1000-3000ms
Depth	0-4mm
Net Weight	22Kg
Voltage	AC220V±10% 50HZ, 110v±10% 60HZ

- Face Lifting
- Wrinkle Reduction
- Skin Tightening
- Pore Reduction
- Acne Scars
- Stretch Marks

Treatment Scope




Before After



Before After



Before After

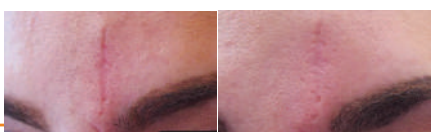

Before After



Before After



Before After

KN - 5000E

308nm Excimer System

LATEST DOUBLE HANDLES DIODE LASER HAIR REMOVAL

High-intensity UV light makes fast treatment effect, and reduce the patient's cycles and times. Suitable for small areas, good targeting. Convenient treatment handle can precisely control the radiation range. Small Size, easy to carry One-key control the light source output, safer treatment. Built-in specific UV dose plan, MED test function, and make the operation easier. The two treatment modes of dose and time can be selected by the user at will. Multi-specification treatment handle hood, users can choose according to the size of the treatment area to avoid unnecessary radiation output. The constant current and constant voltage design is adopted to ensure the stability and uniformity of the spot. 308 excimer power is 10 times bigger than normal uvb lamp.



SPECIFICATIONS

Light Source	XeCl 308nm Excimer
Light Intensity	35mW/cm ²
Irradiation Area	22cm ² ± 10%
Peak Wavelength	308nm ± 2nm
Display Screen	2.2 inch
Energy Density	Upto 5000 mj cm ²
Working Mode	Time and Dose
Treatment Record	Yes
MED Test Function	Yes
Light Source Life time	≥500 hours
Time setting range	0-120S
Weight	1089g
Power Supply	AC220V±10% 50Hz±2%



Vitiligo



Eczema



Psoriasis



Pityriasis Rosea

80-1B Electric Centrifuge Machine

Working Principle

This led light therapy facial mask can supplement the skin's needs, strengthen the muscle base, help the follow-up absorption and maintenance to say goodbye to the bulky pores, truly enhance the skin's firmness, restore the skin's elasticity, smoothness, tenderness and luster, nourish and moisturize the skin from the inside out.

Technical Parameters

Power Supply	AC 220V/110V 50Hz/60Hz
Max Speed	4000rpm
Capacity	20ml*6 (angle rotor of 45 degree)
Max rcf	1790xg
Timer	0-60min
Net Weight	06Kg
Motor Power	40W



Features

80-1B the electric centrifuge is a centrifugal separation test instrument, using centrifugal force to separate the material in the solution. The instrument adopts stepless speed regulation and automatic adjustment of the balance device, which has a stable running, small size, beautiful shape.

The advantage of low temperature rise, high efficiency and wide applicability special aluminum alloy centrifugal turnable to ensure that the centrifuge tube, not easily damaged.

Wide Application

Multipurpose



Medical



Fat Purification



Research



Serum Isolation



Drug Development



Liquid Separation



Latest Diode Laser Hair Removal

Comfortable | Safety | Effective

The principle of diode laser hair removal treatment is based on selective photothermal decomposition. The melanosomes in the hair follicle can selectively absorb the energy of the laser. The laser energy emitted by the machine is easily absorbed by the colored hair follicle without damaging the epidermal tissue. The energy emitted by the laser is absorbed by the hair and the pigment in the hair follicle. It is converted into heat, thus increasing the temperature of the hair follicle. When the temperature rises to a certain level, the hair follicle will be irreversibly damaged, and the hair will lose its original environment and be completely removed.

Specifications

Power Supply	3000W
Power of Handle	600-2000W optional
Wavelength	808nm (755nm-808nm+1064nm optional)
Machine Screen	12.1 inch
Handle Screen	1.54 inch
Energy Density	1-120J/cm ² (Deviation $\leq\pm 2\%$)
Pulse width range	1-300ms (600W) 1-200ms (1200-2000W)
Spot Size	12*12mm ; 12*20mm ; 12*24mm ; 12*28mm
Frequency	1-10HZ (1-20hz 2000w)
Cooling System	TEC cooling system
Net Weight	40kg
Dimension	400*530*440mm
Package Size	740*580*700mm
Fuse specification	$\varnothing 5 \times 25$ 10A
Voltage	AC220V $\pm 10\%$ 10A 50HZ , 110v $\pm 10\%$ 10A 60HZ



Latest Four Wavelength Diode Laser

Comfortable | Safety | Effective | Fast


The principle of diode laser hair removal treatment is based on selective photothermal decomposition. The melanosomes in the hair follicle can selectively absorb the energy of the laser. The laser energy emitted by the machine is easily absorbed by the colored hair follicle without damaging the epidermal tissue. The energy emitted by the laser is absorbed by the hair and the pigment in the hair follicle. It is converted into heat, thus increasing the temperature of the hair follicle. When the temperature rises to a certain level, the hair follicle will be irreversibly damaged, and the hair will lose its original environment and be completely removed.

Technical Parameters

Laser Type	Semiconductor laser
Power	2500W
Power of Handle	1200W
Wavelength	755nm+808nm+940nm+1064nm
Energy Density	1-120J/cm ² (Deviation≤+2%)
Pulse Duration	10ms-200ms
Spot Size	12*20 mm, Facial Tip Ø8
Frequency	1-10HZ
Cooling System	TEC cooling system
Fuse Specification	Ø5×25 10A
Voltage	AC220V±10% 10A 50HZ, 110v±10% 10A 60HZ



PRINCIPLE OF TREATMENT

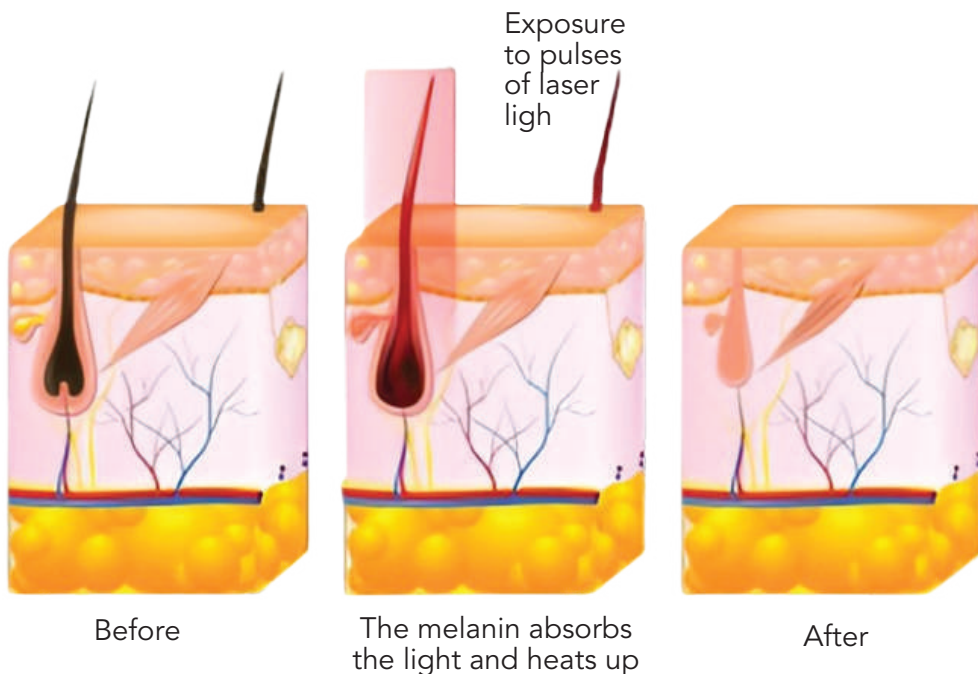

Veloxis Pro Diode Hair Removal

Dual Channel Comfortable Cooling Plate

TEC (Thermo-electric cooling) cooling system, the best stable cooling system, is a perfect match for the diode laser machine in a more suitable working environment, protecting laser bar and increasing the continuous working period of machine to **24 Hours continuous work**, even preventing the risk of combustion. Combined with **sapphire contact cooling** and **TEC cooling**, the handle with powerful cooling plate can achieve superior cooling for **comfortable and pain free treatments**.

SPECIFICATIONS

Power	3000w
Output Power	2000W
Wavelength	808nm 755nm 940nm 1064nm
Energy Density	10-60 J/cm ² (Deviation<+2%)
Pulse Width Range	1-200ms
Spot Size	Ø8, 5*5mm, 10*10mm Standard; 12*20mm, 20*40mm
Frequency	1-20HZ
Cooling System	SystemCompressor Cooling System
Voltage	AC220V±10% 10A 50Hz, 110V±10% 10A 60Hz



Multi Spot Sizes




2000w High Power



Super Short Pulse Duration



- * 2000W HIGH POWER
- * AMERICAN COHERENT LASER BAR
- * 100 MILLION SHOTS WARRANTY

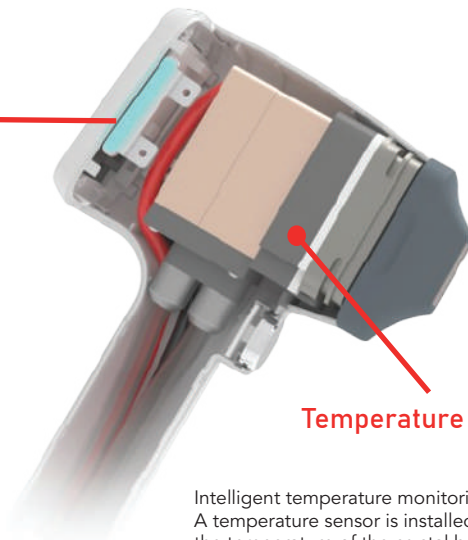

Temperature Sensor Position

Spot Size
20mm*40mm



Intelligent temperature monitoring system
A temperature sensor is installed in the handle, which can keep the temperature of the crystal between 0-4 °C to ensure safe and comfortable treatment



PIORIS 20IV Yag Laser

Dual Channel Comfortable Cooling Plate

Pioris is the first of a new generation of aesthetic lasers that don't rely solely on heat to burn or melt away unwanted tattoo ink or melanin, the pigment that causes dark spots in your skin. Instead, the engineers used cutting edge physics and medical science to help practitioners treat patients in a new way. Instead of building up heat, Pioris delivers energy so rapidly (in trillionths of a second) that tiny particles that make up pigment and tattoo ink vibrate and shatter, without burning surrounding tissue. Less heat means less tissue damage and discomfort.

SPECIFICATIONS

Power	3000w
Wavelength	1064nm, 532nm (585nm, 650nm optional)
Screen	10.4 color touch display
Light Guiding System	7 Joint Arms
Energy	600mj (1064nm), 300mj (532nm)
Frequency	1Hz ~10Hz
Spot Size	2mm ~10mm
Pulse Width	450ps
Aiming Beam	Diode 655nm (RED)
Beam Profile	Top Hat Beam
Voltage	AC220V±10% 10A 50Hz, 110V±10% 10A 60Hz

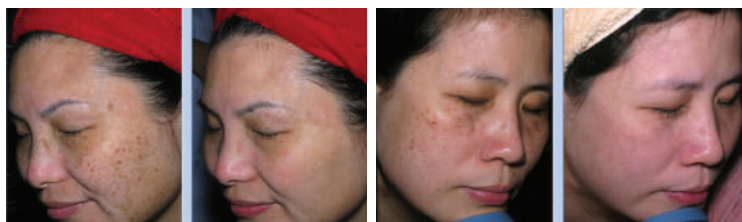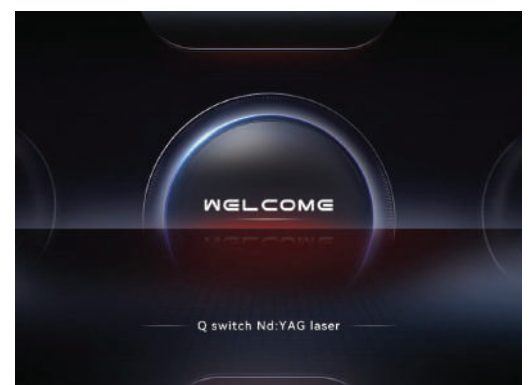


Portable SMAS Focus III 3D HIFU

Face Lifting | Wrinkle Removal | Body Contouring

Pioris is the first of a new generation of aesthetic lasers that don't rely solely on heat to burn or melt away unwanted tattoo ink or melanin, the pigment that causes dark spots in your skin. Instead, the engineers used cutting edge physics and medical science to help practitioners treat patients in a new way. Instead of building up heat, Pioris delivers energy so rapidly (in trillionths of a second) that tiny particles that make up pigment and tattoo ink vibrate and shatter, without burning surrounding tissue. Less heat means less tissue damage and discomfort.

SPECIFICATIONS

Power	800w
Frequency	4MHz
Width	1- 10 mm Adjustable
Length	5 - 25 mm Adjustable
Energy Output	0.2 - 2J/cm2
Cartridge Shot	20000 lines
Cartridges	1.5mm,3.0mm,4.5mm,8.0mm (Standard) 6.0mm, 10.0mm,13.0mm, 16.0mm (Optional)
Net Weight	25kg
Dimension	400*350*600mm
Voltage AC	100-240v 50/60Hz

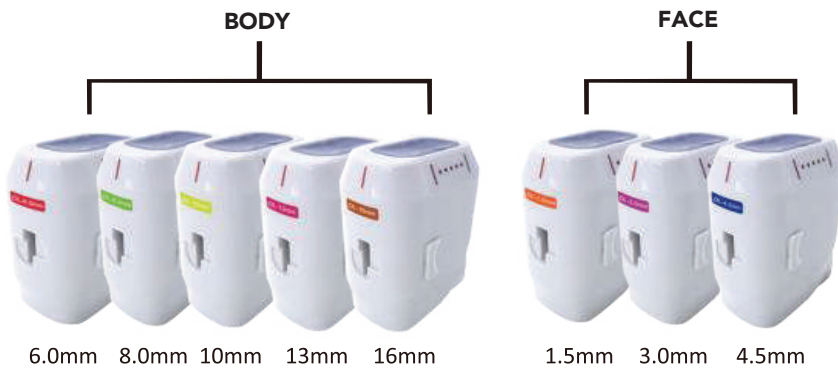

L Procedure Time
30 - 45 Minutes

✓ Back To Work
Immediate

🪡 Anaesthetic
None

♥ Full recovery
Immediate

📌 Sensitivity Period
Upto 48 hours

📈 Results
After 2 - 3 months

+ Number of Treatments
1 every 12 - 18 months

⚠️ Risks & Complications
Possible mild redness, swelling and tenderness post treatment

i Duration Of Results
12 - 18 months

Private Vaginal Tightening Thermiva

No Surgery, No Downtime, No Anesthesia

- Comfortable, In-office treatment
- Three 30-minutes sessions, one month apart.
- Resume sexual activity the same day of the procedure
- Noticeable improvement after 1st or 2nd session
- Long lasting results

SPECIFICATIONS

Input Power	100-120/200-240N+_10%,50/60H
Rated Power Input	160WA
Output Power	20 watts(maximum)50-300 Q
Maximum Output	80V, rms
Operating Frequency	460khz(+_5khz)
Power Delivery Modes	RF Mode. BioMed
Set Temperature	35-40 degree
Fuses	Dual fuses: 3.0 A/250 V
Waveform	Sine wave for RF Mode Square wave for Stimulate Modes
Dimensions	5.75"H * 12.5"W *13.0"D (146 MM * 318 mm *330 mm)
Weight	12.6lbs(5.7kg)
Protection	Class I, Type BF Applied part-defibrillator proof, IXPO, continuous operation. This equipment is not suitable for use in the presence of a flammable anesthetics mixture with air oxygen, or nitrous oxide
Controls	Line Powers ON/OFF, RF On/Off, Stimulate Output, front Panel soft keys (Start. Set Temp. Set Profile, Reset, Width, Frequency and mode)
Display	Impedance, Actual Temp. Elapsed Time, Paused, Volts, Frequency
Connections	Footswitch connector, power cord connector, 8-pin device connection port, neutral electrode connection port, RS-232 connector
Model No	A33



Tighten the inner vagina walls

Increase vagina moisture

Improves Vaginal Looseness

Strengthens vagina muscles

Application




Improves Vaginal Strength

Improves painful intercours

Reduces Urinary Leakage

Improving the tightness of Vaginal and Skin Rejuvenation

Our Prestigious Clients

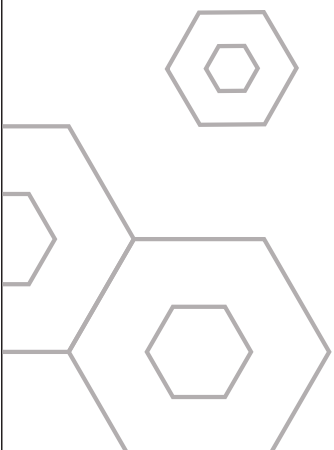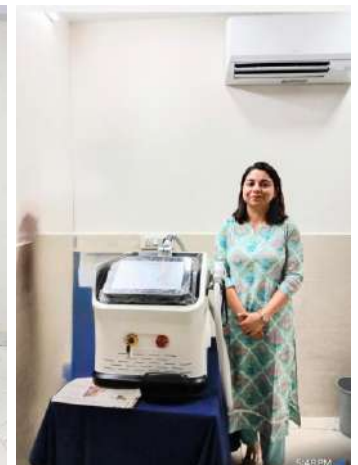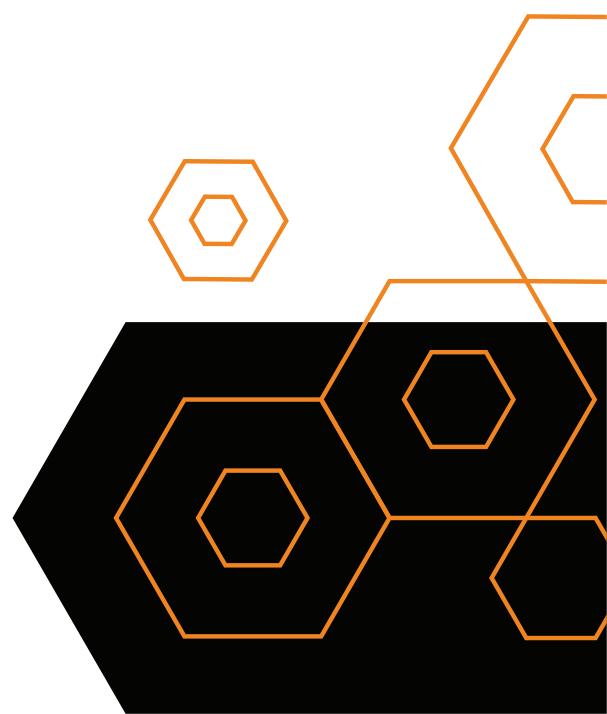

Contact Us

Wave City, Ghaziabad, UP - 201010



+91 7303111897 / 898



www.aesthetinext.com



info@aesthetinext.com

